

Curettage

For 25 years Bionix has been advancing the technology of cerumen removal. Each innovation has added value to every procedure making Bionix the World Leader in Safe Cerumen Removal. Bionix continues that tradition with the introduction of the Articulating Safe Ear Curette™ - combining the safety of the original Safe Ear Curette with an innovative articulating tip.

Features and Benefits

The Articulating Safe Ear Curette™ is the only curette that articulates to assist in the removal of cerumen from the ear canal. The Articulating Safe Ear Curette works because of the unique living hinge at the tip and the split handle design.

Once positioned beyond an obstructive cerumen plug, usually through a space between the ear canal wall and the plug, the tip can be articulated by simply sliding the forefinger and thumb in opposite directions. The cerumen can then be pulled straight from the ear canal without using a blind sweeping motion.

- *Articulating tip eliminates need for top down sweeping motion that can abrade the ear canal.*
- *Ability to work behind cerumen plug or foreign body.*
- *Two part sliding body for easy manipulation.*
- *Firm, articulating tip to make tough cerumen removal easier.*
- *Single-use to eliminate the time, labor, and costs associated with re-sterilizing steel curettes and reducing the risk of any cross contamination.*
- *Made in the U.S.A.*

The Articulating Safe Ear Curette tip has a 90° range of motion.



The Articulating Safe Ear Curette tip is controlled by sliding the body in opposite directions using your fingertips.

Product Number



#8111
Articulating
VersaScoop™

(50 Articulating Ear Curettes Per Box)

Curettage can be billed under CPT code 69210 for potential reimbursement.



Patent Pending
©2010 Bionix Medical Technologies

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.6810 • Fax: 800.455.5678
www.BionixMed.com

