

Enteral Feeding Tube Clearing

In long-term care, home health care, and hospital environments where various enteral nutrition therapies are practiced, the inevitable clogging - over time - of enteral feeding tubes has created problems for both patients and care givers. Recognizing the problem, Bionix developed a solution using a basic screw thread design. Today there is a full line of Bionix Enteral Feeding Tube DeCloggers that can quickly and effectively bore through and dislodge blockages.

Features and Benefits

- *Clears tubes in as little as 2 minutes eliminating long interruptions in your patients feeding or medication schedule*
- *Reduces need for costly tube replacement.*
- *Safe, flexible plastic design that conforms to the feeding tube, greatly reducing the risk of puncture.*
- *Available in five unique sizes and lengths to fit different feeding tubes.*
- *Features safety stop disk to prevent over-insertion.*
- *Packaged 10 DeCloggers per box.*
- *Made in the U.S.A.*



BIONIX[®]
MEDICAL TECHNOLOGIES


Phone 1.800.551.6810

Fax 1.800.455.5678

Web www.BionixMed.com

Enteral Feeding Tube DeCloggers®

DeClogger offers many advantages when compared to the other methods that are used to clear obstructed G & J feeding tubes. It's safer than current methods being performed such as metal guide wires, fluid-filled syringes, cranberry juice, soda and enzyme mixtures. An Enteral Feeding Tube DeClogger is effective, safe and clears a tube in less than 2 minutes to improve patient care.

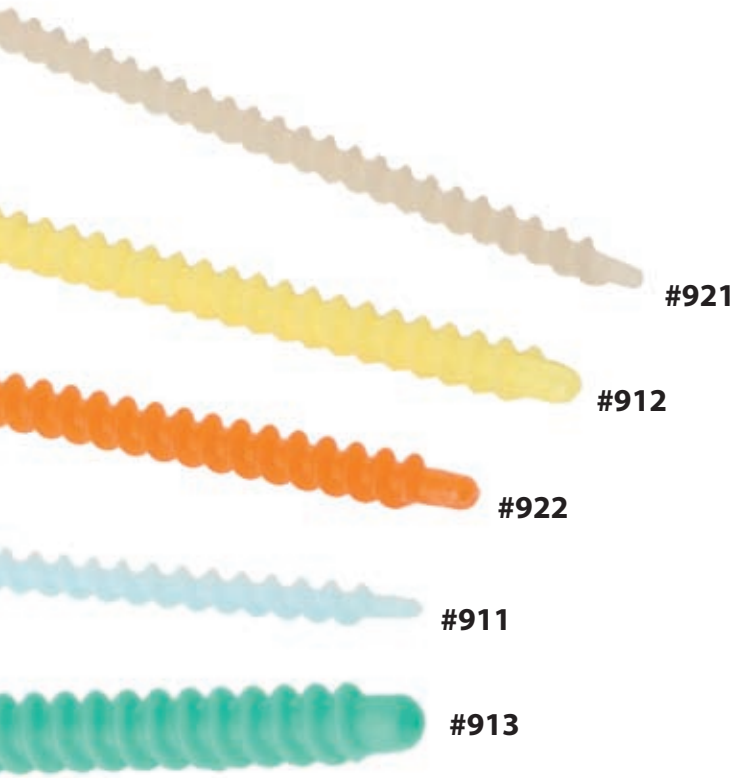

Gastrostomy or Jejunostomy

Product Number	Product Size	Length
911 – Blue	14 – 16 French	39.5 cm
912 – Yellow	16 – 18 French	39.5 cm
913 – Green	20 – 22 French	39.5 cm

Peg Type or Replacement Type

Product Number	Product Size	Length
921 – Brown	14 – 16 French	21.5 cm
922 – Orange	18 – 24 French	21.5 cm

DeCloggers are packaged 10 per box.

How to Use

	<p>Over time, residue from enteral solutions and medications clog the tube, putting the patient at risk.</p>		<p>The DeClogger is inserted down the tube where slight twisting action is applied and the build-up or clog is dislodged and cleared.</p>		<p>Within two minutes, the build-up or clog can be cleared and the tube restored to its normal free-flowing condition.</p>
--	--	--	---	--	--



©2010 Bionix Medical Technologies

5154 Enterprise Blvd. • Toledo, Ohio 43612
 Phone: 800.551.6810 • Fax: 800.455.5678
www.BionixMed.com

