

Lighted Forceps for Foreign Body Removal

The Bionix Lighted Forceps for foreign body removal is an innovative new tool that provides primary and acute care physicians with light and magnification in a small, single-use forceps for the ear and nose. The Lighted Forceps is an extension of the innovative, award winning Lighted Ear Curette with Magnification product line.

Features and Benefits

- *Visualization* – A brilliant white light projected to the tip of the forceps illuminating the orifice and the foreign body.
- *Safety* – Illumination and magnification improves procedural accuracy, avoiding the risk of injury from a blind procedure.
- *Efficacy* – Forceps open to accommodate object as large as 5 mm with gripper teeth for a secure hold.
- *Simplicity* – Simple spring grip design for easy, single handed operation.
- *Convenience* – Eliminates the need to use a separate light source or magnification device in an attempt to enhance visualization.
- *Variety* – Ideal for the nose or ear and a variety of foreign bodies.
- *Made in the U.S.A.*



Product Offering

2750 - Box of 10 Forceps, 1 Light Source, 1 Magnification Lens

2775 - Box of 10 Replacement Forceps

2200 - Extra Light Source

2265 - Pack of 5 Magnification Lenses

How It Works

The forceps tip works by squeezing the spring grip that is positioned just below the attachment point of the light source. The spring grip is fixed to the clear forceps tip near the light source attachment by means of a retaining ring. When squeezed, the spring grip is forced to slide down towards the forceps tip closing it and providing gripping strength.

CPT Codes

We recommend that you submit CPT code 30300 for potential reimbursement of foreign body removal of the nose and CPT code 69200 for potential reimbursement of foreign body removal of the ear.



Patent Pending
©2010 Bionix Medical Technologies

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.6810 • Fax: 800.455.5678

www.BionixMed.com

