

Ear Irrigation

Over 25 years ago, Bionix invented single-use Safe Ear Currettes™ and subsequently became the world leader in cerumen management technologies. Along the way, we recognized that the use of an ear curette may not be adequate for all cerumen obstructions and that irrigation of the ear canal may be a better treatment option for some patients. While irrigation procedures produce results, there exists a potential risk of injury caused by pressurized liquid directly impacting the tympanic membrane. Bionix began a process to develop a product that effectively eliminated that risk while providing exceptional results.


Features and Benefits

The OtoClear Ear Irrigation Tip with Gentle Touch Design was developed to safely and thoroughly clear the ear canal of wax and debris. The tip features a soft, flared design preventing over-insertion into the ear canal. When activated, the tip sprays divergent streams of water, gently toward the walls of the ear canal, not directly against the tympanic membrane.

The OtoClear Ear Irrigation Tip is designed to conveniently attach to the Portable Waterpik®, The NEW Spray Wash Bottle or a table top Waterpik®.

The OtoClear Ear Irrigation Tip:

- **NEW** Improves patient comfort because of the soft, Gentle Touch Design.
- **NEW** Directs water flow away from the tympanic membrane at a 30° angle for increased efficacy, virtually eliminating the risk of rupture.
- Eliminates over-insertion into the ear canal because of its flared tip design.
- Gently and thoroughly lavages ear canal.
- Provides “real-time” clearance of obstructive cerumen.
- Features exit portals to prohibit pressure buildup and direct drainage into a basin, reducing backsplash and clean-up time.
- Can be billed under CPT code 69210 for potential reimbursement of cerumen removal.

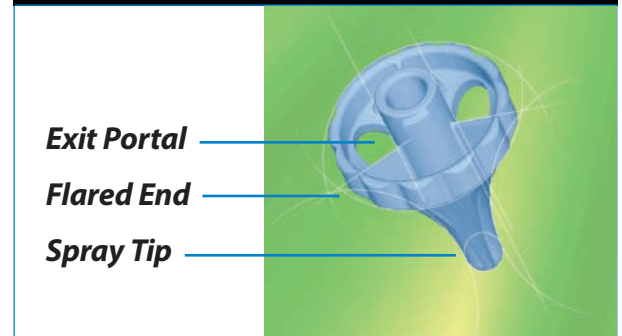

Product Numbers

- #7200 - 40 single-patient tips per box
- #7245 - Portable Waterpik®
- #7280 - Ear Lavage System
- #7290 - OtoClear Spray Wash Kit
- #3700 - Ear Irrigation Basins (box of 3)




BIONIX
MEDICAL TECHNOLOGIES

Phone 1.800.551.6810


Fax 1.800.455.5678

Web www.BionixMed.com


How to Use

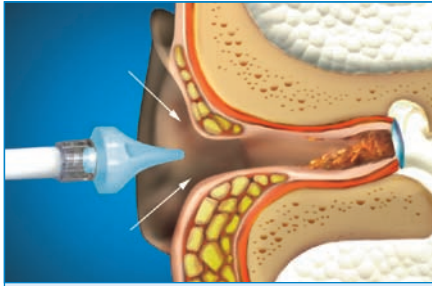

Insert *OtoClear* tip fully into the ear canal. The flared design of the *OtoClear* tip protects against over-insertion.

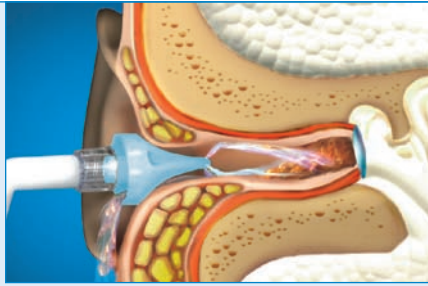

Begin irrigation. Note: The *OtoClear* tip directs the water towards the walls of the ear canal creating a turbulent lavage that effectively dislodges the cerumen. The exit portals control back-splash and deliver the effluent into the basin.

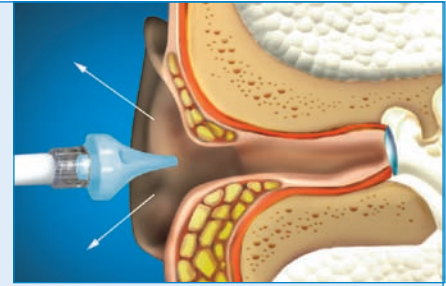

When procedure is complete, remove and dispose of the single-use *OtoClear* tip.

Thoroughly read the complete directions before using the *OtoClear* Ear Irrigation tip. Do not perform irrigation if perforations or tympanostomy tubes are present or suspected. Discontinue irrigation immediately if bleeding, irritation or other trauma to the ear canal or tympanic membrane occurs. Hard or impacted wax may require a cerumenolytic agent.

OtoClear® Ear Irrigation Systems

OtoClear Tips were designed for compatibility with multiple delivery devices.

With each delivery device, *OtoClear* delivers predictable effectiveness and is the ultimate in safe ear irrigation.

#7280

Ear Lavage System

Recommended for high volume facilities

- A completely portable irrigation delivery device that can be used for procedures anywhere in or out of the office
- Provides effective, pulsed irrigation
- Higher volume than manual devices
- One-touch High and Low settings
- Proven safety & effectiveness (*Children's Hospital of Pittsburgh Study*)
- Rechargeable battery
- 210ml self-contained water reservoir

System Includes:

- 1 Portable Waterpik®
- 40 *OtoClear* Tips
- 1 Ear Basin



#7290

OtoClear Spray Wash Kit

Recommended for low volume facilities

- Manual, intermittent lavage
- Delivers greater volume compared to syringes or other manual devices
- Efficient design allows for two-handed procedure

Kit Includes:

- 1 Spray Wash Bottle
- 20 *OtoClear* Tips
- 1 Ear Basin



OtoClear Ear Irrigation Tips
Now with *Gentle Touch* Design

Looking for Safe Cerumen Removal Tips and Techniques? Visit www.BionixMed.com For Your FREE Video!



©2011 Bionix Medical Technologies.
**OtoClear*® Tips - U.S. Patent #6,706,023
Waterpik® is a registered trademark of
Water Pik, Inc. The *OtoClear* Safe Irrigation
Tip is not affiliated with Waterpik Technologies.

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.6810 • Fax: 800.455.5678
www.BionixMed.com

