

Curettage

For years doctors used stainless steel ear currettes to remove wax in the ear canal that obstructed their view of the tympanic membrane. The pain and trauma caused by these hard metal currettes prompted Bionix to develop the first Safe Ear Curette™, the White FlexLoop®, which quickly became the standard of care for curettage. Today, Bionix offers a complete line of seven different currettes each designed for a specific type of patient and cerumen condition.

Features and Benefits

- Flexible to reduce and avoid injury to the ear canal or tympanic membrane, making them safer than stainless steel currettes.
- Single-use to eliminate the time, labor and costs associated with re-sterilizing steel currettes as well as reduce the risk of any cross contamination.
- Color-coded for easy selection.

The complete line of Safe Ear Currettes consists of seven specially designed ear currettes which are the:

- **White FlexLoop®**
- **Blue InfantScoop®**
- **Green MicroLoop®**
- **Red AngleLoop®**
- **Yellow CeraSpoon®**
- **Purple VersaLoop®**
- **Orange ControlLoop®**

Each curette has been engineered and manufactured to meet the exacting quality and safety standards for Bionix Safe Ear Currettes. For over 18 years this has meant:

- *In-house product development*
- *Multiple precision measurement checks*
- *100% in-process inspection of currettes during packaging*
- *FDA compliant polymers and colorants*



50 single-use currettes per box

Curettage can be billed under CPT code 69210 for potential reimbursement.



BIONIX
MEDICAL TECHNOLOGIES

Phone 1.800.551.6810

Fax 1.800.455.5678

Web www.BionixMed.com

Safe Ear Curettes™

Bionix developed the Safe Ear Curette to reduce the pain and trauma caused by hard, metal curettes over 20 years ago. Since then they have become the brand that Medical Professionals rely on and trust to provide a high quality Safe Ear Curette. Remember, if it doesn't say Bionix on the handle, you don't have a Safe Ear Curette in your hand.

General



9555 White FlexLoop®

The White FlexLoop is an all purpose curette designed for normal curettage in a variety of patients. 4mm

6333 Yellow CeraSpoon®

The Yellow CeraSpoon is perfect for softer wax, especially in patients who produce excessive amounts of cerumen. 4mm

2999 Orange ControLoop®

The Orange ControLoop has a bent "Shapleigh" style tip that allows the wax to be pulled straight out of the ear canal. 4mm

3444 Red AngleLoop®

The Red AngleLoop has an angled tip that facilitates reaching the hard-to-get, tough cerumen common in older patients. 4mm

4111 Purple VersaLoop®

The Purple VersaLoop has a smooth flexible tip that gives the user exceptional control while offering the patient more comfort. 3mm

4888 Blue InfantScoop®

The Blue InfantScoop is designed for smaller ears and may be used through an otoscope. 2mm

1222 Green MicroLoop®

The Green MicroLoop is more rigid in construction and is designed to remove tough, impacted cerumen. 3mm

Adult

Youth

How to Use



Normal Curettage

1. Use otoscope to determine cerumen location.
2. Choose the appropriate curette for the type of patient and cerumen.
3. Place curette into ear canal and remove cerumen.
4. Repeat otoscope exam and determine location of any remaining cerumen.



Otoscope Curettage

1. Use otoscope to determine cerumen location.
2. Select one of the Youth **Safe Ear Curettes™**.
3. Slide scope magnification to one side.
4. Place curette through otoscope and remove cerumen.

BIONIX, CeraSpoon, FlexLoop, InfantScoop, MicroLoop, AngleLoop and VersaLoop are registered trademarks. Safe Ear Curettes and ControLoop is a trademark of BIONIX. Patented - The ControLoop®, CeraSpoon®, VersaLoop®, FlexLoop®, InfantScoop® and AngleLoop®.



©2010 Bionix Medical Technologies.

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.6810 • Fax: 800.455.5678
www.BionixMed.com

