

Wound Approximation

With the use of tissue adhesives in wound closure procedures gaining wider acceptance around the world, Bionix recognized that many physicians lacked the instruments to get the best results from this technology. Bionix then developed a line of forceps to serve as essential complements to tissue adhesive wound repair. TruLine® Wound Closure Forceps continue to give physicians a better way to approximate a wound.

Features and Benefits

All TruLine Wound Closure Forceps:

- Easily approximate wound edges quickly and effectively.
- Improve the manipulation of simple and complex wounds.
- Provide superior wound visualization for proper adhesive application.
- Proprietary non-stick coating virtually eliminates any sticking problems.
- Allow for easy application in an office setting.
- Reduce procedure time.
- We recommend that you submit this under CPT code 12001 through CPT code 12021 (depending on the length of the laceration) for potential reimbursement.



Product Numbers - *15 Forceps per Box
#9840 - Multi-Pak
#9853 - Purple Pediatric
#9875 - Green Universal
#9869 - Blue Fine



BIONIX
MEDICAL TECHNOLOGIES

Phone 1.800.551.6810

Fax 1.800.455.5678

Web www.BionixMed.com

TruLine® Wound Closure Forceps

TruLine Wound Closure Forceps are designed to give physicians greater control and enhanced visualization during specific wound closure procedures. Some physicians have also found that TruLine forceps work exceptionally well when using surgical staples or other wound closure techniques.



#9869 Blue Fine

- For small trocar and cosmetic closures
- 7 mm wide tip



#9853 Purple Pediatric

- Round tips that provide more comfort to pediatric patients
- 14 mm wide tip



#9875 Green Universal

- For closure of large, complex wounds featuring teeth for wound debridement.
- 17 mm wide tip

How to Use



Apply downward pressure to grip the skin surface.



Gently close TruLine forceps to approximate wound edges.



Apply tissue adhesive according to manufacturer's instructions.



Patented. ©2010 Bionix Medical Technologies

5154 Enterprise Blvd. • Toledo, Ohio 43612
Phone: 800.551.6810 • Fax: 800.455.5678

www.BionixMed.com

